




QUICK-START FLAG GUIDE FOR THE ARGOS CARDIAC OUTPUT MONITOR


1. To Activate % Change:
 - a. Touch desired parameter (CO, SV, etc.).
 - b. Touch “Show % change” and ensure it is marked with a .
 - c. Touch “Save”.

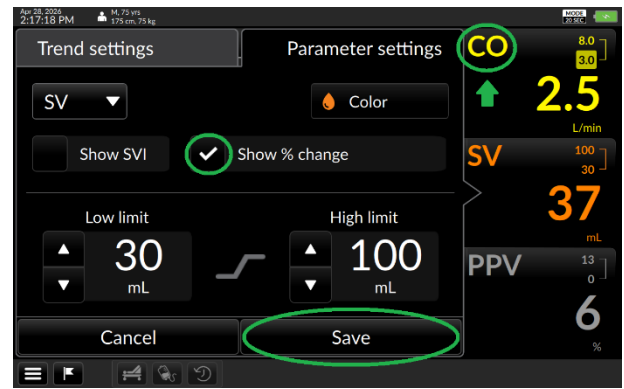


2. A % number will appear next to the relevant parameter(s) indicating the % increase or decrease in the measured value since the most recent flag.

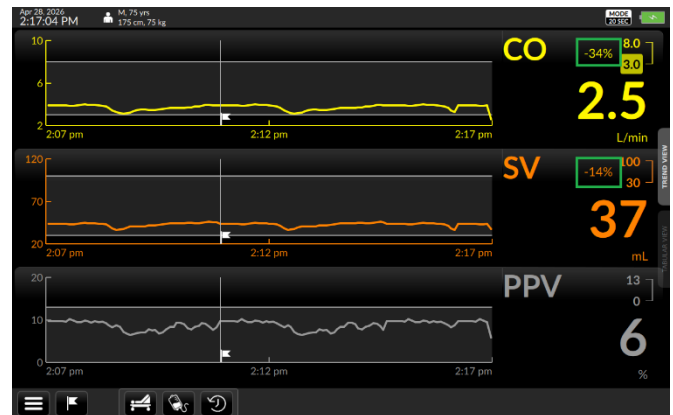

3. To Flag an Intervention (BOLUS, PLR, etc.):
 - a. Touch **Flag**  icon at bottom left of screen.
 - b. Use onscreen keyboard to describe flagged event.
 - c. Touch “Save” - a flag will post on all trend screens and % change will update and reflect % change since most recent flag.
 - d. This is appropriate for entering either the Start or End of interventions.

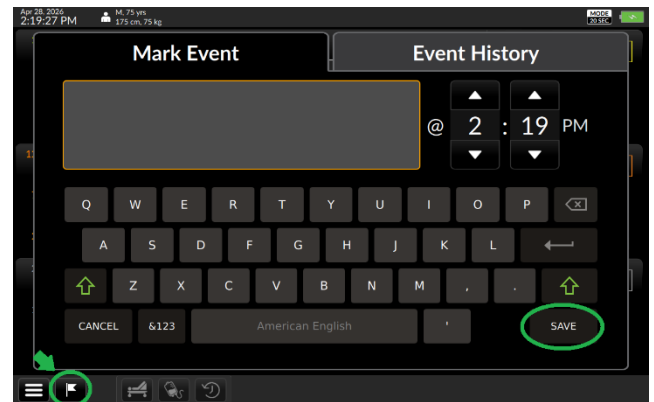


4. To View Trended Values:
 - a. Touch and hold finger on the trend waveform until gray cursor appears.
 - b. The gray cursor can be dragged back & forth along the trend waveform to view specific parameter values and flagged text.

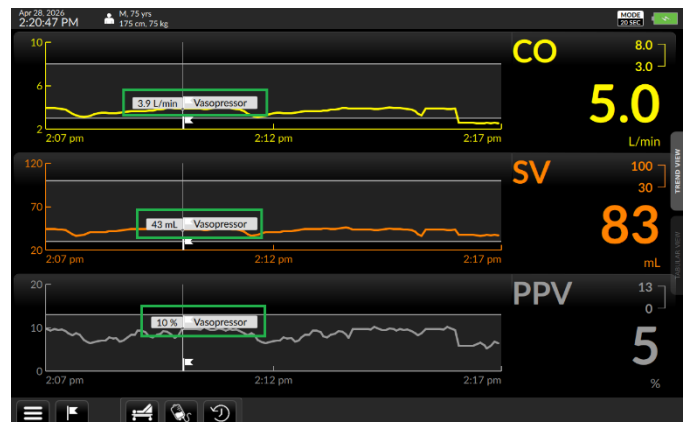

QUICK-START FLAG GUIDE FOR THE ARGOS CARDIAC OUTPUT MONITOR

5. To Review or Edit a Flagged Intervention/Event:

- Touch .
- Touch “**Event History**” & select the specific event to review/edit. The event time and/or description can then be edited.
- Touch “**Save**” to keep changes.

